


The cushions shape exactly to the body suplying stability, support, comfort and pain relief.

Silent material – no noise

No Deformation of tissue:

Anti-decubitus - perfect pressure distribution for prevention and healing of pressure sores

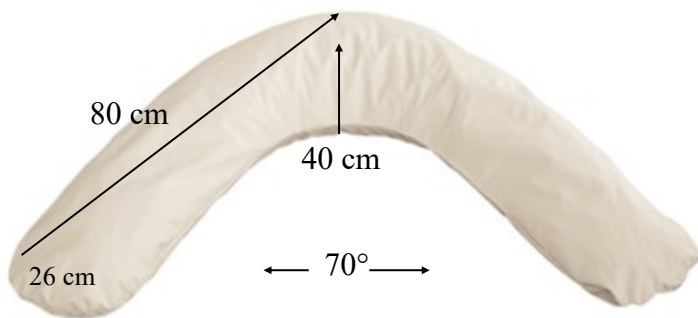

The Pressure Distributing Positioning System

Type 10, 11 cushions

No. 10.00	15 x 15 cm
No. 11.00	18 x 30 cm
No. 11.01	30 x 40 cm
No. 11.02	40 x 50 cm
No. 11.03	40 x 60 cm
No. 11.04	40 x 75 cm
Height app. 7/8 cm	

Type 22 cushions

No. 22.01	50 x 60 cm
No. 22.02	50 x 90 cm
No. 22.03	60 x 90 cm
No. 22.04	65 x 100 cm
Height app. 11 cm	

Type 33 cushion

No. 33.01	Boomerang
160 cm	
26 (40) cm	
Height 10 cm	

Type 44 cushions

Long Cushions	
No. 44.01	160 x 30 cm
No. 44.02	180 x 30 cm
No. 44.04	220 x 30 cm
Height 13 cm	

No. 66.01 Neck cushion,
33x29x12 cm

No. 77.00 Neck Roll
25 cm Ø12 cm



Filling is 100% viscoelastic foam granulate: SAFE ViscoPore / Fabric material is 100% Cotton
Three types of covers for Harmony: Comfor Incontinence covers, Soft-Cell covers, Cotton velour covers.


Machine wash up to 95 ° C and tumble dry. Optimal hygiene and cleanliness can be maintained.

Washing removes bacteria, urine odor, sweat odor and perfume odor.

SAFE-MED
 Pressure Distributing Products

Type 33.01 "Boomerang"
 160 x 26/40 cm 70°

Comfortable and lower back supportive position while seated.

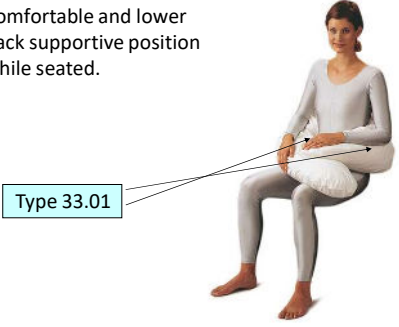

Type 33.01

1

SAFE-MED
 Pressure Distributing Products

Type 33.01 / "Boomerang"
 160 x 26/40 cm 70°

Comfortable dorsal position with full upper body support/stabilisation



Type 33.01


2

SAFE-MED
 Pressure Distributing Products

Type 33.01 "Boomerang"
 160 x 26/40 cm 70°

Comfortable dorsal position with full upper body support/stabilisation and leg support with pressure relief of heels

Type 22.02, 50 x 90 cm
Type 22.03, 60 x 90 cm
Type 22.04, 65 x 90 cm



Type 33.01 Type 22.02, 22.03 or 22.04

3

SAFE-MED
 Pressure Distributing Products

Type 33.01 "Boomerang"
 160 x 26/40 cm 70°

Comfortable side position with upper body support/stabilisation.
 Head support and relief of the arm.




Type 33.01

4

SAFE-MED
 Pressure Distributing Products

Type 33.01 "Boomerang"
 160 x 26/40 cm 70°

Comfortable side position with stable back support/stabilisation.
 Integrated neck support + knee pad.

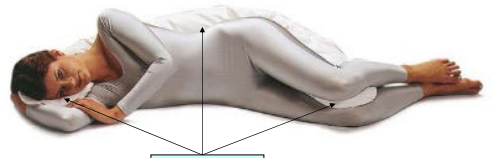

Type 33.01


5

SAFE-MED
 Pressure Distributing Products

Type 33.01 "Boomerang"
 160 x 26/40 cm 70°

Comfortable side position with back stabilisation, neck support and additional knee padding.

Type 11.01, 30 x 40 cm
Type 11.02, 40 x 50 cm
Type 11.03, 40 x 60 cm
Type 11.04, 40 x 75 cm




Type 33.01 provides head support and back support Type 11.01, 11.02, 11.03, 11.04 Knee protection

6

SAFE-MED
 Pressure Distributing Products

Comfortabel dorsal position with neck support, side stabilisation, leg support and relief of heels.

Type 11.01, 30 x 40 cm
Type 11.02, 40 x 50 cm
Type 22.04, 65 x 100 cm




Type 11.02 Type 11.01 Type 22.04

7

SAFE-MED
 Pressure Distributing Products

Comfortable dorsal position with neck support. Dedicated knee support and heel padding.

Type 11.02, 40 x 50 cm
Type 11.03, 40 x 60 cm
Type 11.04, 40 x 75 cm
Type 22.01, 50 x 60 cm
Type 44.01, 160 x 30 cm




Type 22.01 Type 44.01 folded Type 11.02, 11.03, 11.04

8

SAFE-MED
 Pressure Distributing Products

Comfortable side position with neck support and knee protection.

Type 11.01, 30 x 40 cm
Type 11.02, 40 x 50 cm
Type 11.03, 40 x 60 cm
Type 22.01, 50 x 60 cm




Type 11.02, 11.03 or 22.01 Type 11.01, 11.02 or 11.03

9

SAFE-MED
 Pressure Distributing Products

Comfortable side position with upper and lower body support. Providing full support from head to ankle.

Type 44.01, 160 x 30 cm
Type 44.02, 180 x 30 cm
Type 44.04, 220 x 30 cm




Type 44.01, 44.02 or 44.04

10

SAFE-MED
 Pressure Distributing Products

Comfortable side position with front support. Head support, stomach support and support of bended knee. Separat ankle support.

Type 44.01, 160 x 30 cm
Type 44.02, 180 x 30 cm
Type 44.04, 220 x 30 cm
Type 11.01, 30 x 40 cm




Type 44.01, 44.02 or 44.04 Type 11.01

11

SAFE-MED
 Pressure Distributing Products

Comfortable side position with stability and the feeling of safety. Head support. Front support with elbow support and protection between legs and ankles.

Type 11.02, 40 x 50 cm
Type 22.01, 50 x 60 cm
Type 44.01, 160 x 30 cm
Type 44.02, 180 x 30 cm
Type 44.04, 220 x 30 cm



Type 22.01 or 11.02 Type 44.01, 44.02 or 44.04

12